

Automatic Upper Arm Blood Pressure Monitor

M3 Comfort (HEM-7155-E)

X3 Comfort (HEM-7155-EO)

Read Instruction manual ① and ② before use.

- FR** Lire le mode d'emploi ① et ② avant l'utilisation.
- DE** Lesen Sie vor der Verwendung die Gebrauchsanweisung ① und ②.
- IT** Leggere il manuale di istruzioni ① e ② prima dell'uso.
- ES** Lea el manual de instrucciones ① y ② antes del uso.
- NL** Lees de gebruiksaanwijzing ① en ② voor gebruik.
- RU** Прочтите руководство по эксплуатации ① и ② перед использованием.
- TR** Kullanmadan önce, kullanım kılavuzu ① ve ②'yi okuyun.

AR اقرأ دليل الإرشادات ① و ② قبل الاستخدام.

EN
FR
DE
IT
ES
NL
RU
TR
AR
Symbols

Symboles / Symbole / Simboli / Símbolos /
Symbolen / Символы / Semboller / الرمز

All for Healthcare

CE0197

EN

1. Introduction

Thank you for purchasing the OMRON Automatic Upper Arm Blood Pressure Monitor. This blood pressure monitor uses the oscillometric method of blood pressure measurement. This means this monitor detects your blood movement through your brachial artery and converts the movements into a digital reading.

1.1 Safety Instructions

This instruction manual provides you with important information about the OMRON Automatic Upper Arm Blood Pressure Monitor. To ensure the safe and proper use of this monitor, READ and UNDERSTAND all of the safety and operating instructions. **If you do not understand these instructions or have any questions, contact your OMRON retail outlet or distributor before attempting to use this monitor. For specific information about your own blood pressure, consult with your physician.**

1.2 Intended Use

2.1 Warning

Indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

- DO NOT use this monitor on infants, toddlers, children or persons who cannot express themselves.
- DO NOT adjust medication based on readings from this blood pressure monitor. Take medication as prescribed by your physician. ONLY a physician is qualified to diagnose and treat high blood pressure.
- DO NOT use this monitor on an injured arm or an arm under medical treatment.
- DO NOT apply the arm cuff on your arm while on an intravenous drip or blood transfusion.
- DO NOT use this monitor in areas containing high frequency (HF) surgical equipment, magnetic resonance imaging (MRI) equipment, computerized tomography (CT) scanners. This may result in incorrect operation of the monitor and/or cause an inaccurate reading.
- DO NOT use this monitor in oxygen rich environments or near flammable gas.

Consult with your physician before using this monitor if you have