GB INSTRUCTION FOR USE

Read the instruction for use before using the product. Store at room temperature. Avoid storing product in direct sunlight and/or high temperatures. Use in well-ventilated areas. External use only.

1. CONTENTS

1x Instruction for use
1x Aerosol dispenser contains 200 ml

CONTAINS

Butane, Ethyl acetate, Synthetic resin, Propane, Isobutane.

2. DESCRIPTION

Pre Tape improves the adhesion of sports tape, pre-wraps, elastic bandages or windings and acts as a connective layer between the skin and (sports) tape. It dries quickly and ensures a durable optimal fixation.

FEATURES

- dries quickly
- hygienic in use
- water- & sweat resistant
- spray can be used up-side-down
- contact spray for better adhesion of various (medical) tapes

ADVICE

Removable with U-Sport Tape Remover or soap and water.

3. INTENDED USE / INDICATIONS FOR USE

The product its indication is to act as a connective layer between skin and various (medical) tapes/bandages. This product is intended for use on:

- healthy people over 12 years of age
- external use only

This product should only be used on undamaged, non-irritated skin.

4. INSTRUCTIONS FOR USE

1. Shake well before use.
2. Spray the product, from a distance of 5-8 cm, in a circular motion on the skin.
3. Apply a thin layer and leave it to dry for 30 seconds.
4. Thereafter apply the (medical/sports) tape following its own instruction for use.

Use in well-ventilated areas. Stop using the product immediately when skin irritation occurs.

5. WARNINGS & PRECAUTIONS

DANGER Extremely flammable aerosol. Pressurised container: May burst if heated. Causes serious eye irritation. May cause drowsiness or dizziness. Repeated exposure may cause skin dryness or cracking. Buildup of explosive mixtures possible without sufficient ventilation.

Read label before use. If medical advice is needed, have product container or label at hand. Keep out of reach of children. Avoid breathing dust / fume / gas / mist / vapours / spray. **IF INHALED:** Remove person to fresh air and keep comfortable for breathing. Call a **POISON CENTER** /doctor if you feel unwell. **IF IN EYES:** Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists: Get medical advice /attention. Use only outdoors or in a well-ventilated area. Store in a well-ventilated place. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Do not spray on an open flame or other ignition source. Do not pierce or burn, even after use. Protect from sunlight. Do not expose to temperatures exceeding 50 °C/122 °F. Dispose of contents / container in accordance with local / regional / national / international regulations.

6. CONTRAINDICATIONS

DO NOT USE

- if sensitive to one of the ingredients
- directly on wounds
- or avoid spraying in the vicinity of the face and/or near the genitalia

When pregnant or breastfeeding consult a doctor before use.

7. POTENTIAL ADVERSE EVENTS

May cause drowsiness or dizziness, consult your physician immediately.

8. COMPLAINTS & CUSTOMER SERVICE SUPPORT

Contact the manufacturer with questions /complaints or to report an adverse reaction.

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