2020/2021

There is no substitute for quality

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Rotary instruments
for podology/footcare
and naildesign

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„There is no substitute for quality“ this conviction has committed our company's philosophy since our foundation in 1905.

Our ideas and all processes of research & development, production and distribution meet with the demands of this philosophy.

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Two exceedingly effective materials - diamond and ceramic - combined to yield an innovative hybrid grit.

The medical premium product series with an internal cooling technique and hybrid grit for working on callus.

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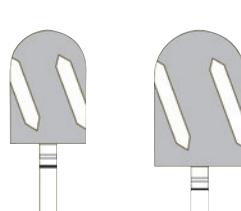
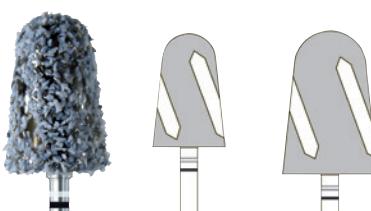
- special arranged aperture slits
- stainless steel material guarantees the disinfection and sterilization persistence
- precise concentricity by one-piece construction
- efficient treatment on callus
- quick removal
- finest surfaces
- less follow-up treatment
- for use in wet- as well as dry technique

super coarse grit
removal of severe callus

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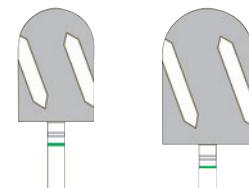
coarse grit
removal of thick callus

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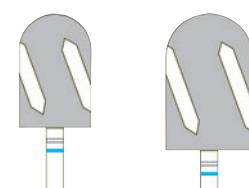
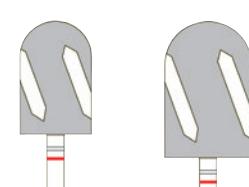
fine grit
removal of incipient callus

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The modern grit - diamond plus ceramic -
is coated on a durable stainless steel cap.

Efficiency:

one HybridCap instead of hundreds.

Sustainability:

instead of throwaway.

Patient friendliness:

smoother surfaces in less time.

Hygiene:

HybridCap and Cap Support are
disinfectable and sterilizable.

A CE-symbol medical product made
by BUSCH

Cap and mandrel have to be disinfected
and sterilized unmounted for hygienic and
technical reasons.

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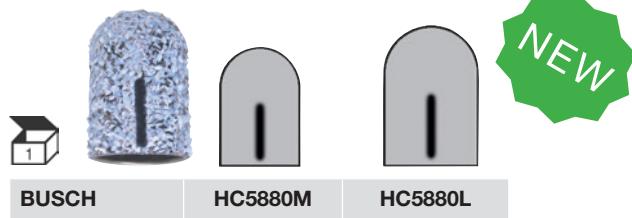
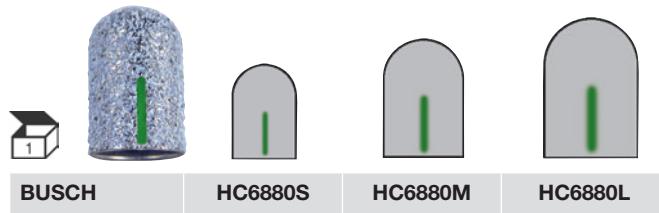
Cap Träger

cap support
support cap
portador cap

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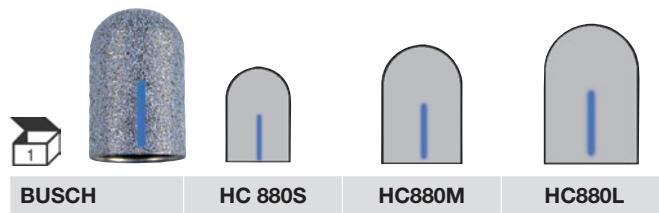
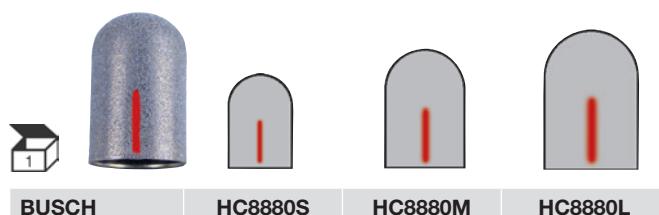
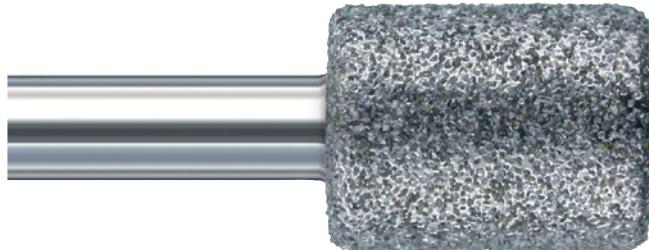
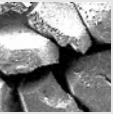
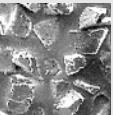
^{*)} The grit size is optimally adjusted to usual applications, speed, sizes and shapes of the diamond instruments but may differ in the mean value for individual instruments.

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The medical product series with an internal cooling technique for working on callus.

- for use in wet- as well as dry technique
- special arranged aperture slits
- stainless steel material guarantees the disinfection and sterilization persistence
- precise concentricity by one-piece construction

mega coarse grit
removal of thick callus

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super coarse grit
removal of hard callus

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coarse grit
removal of incipient callus

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The modern diamond coated series of medical products with CE-symbol for hygienically working on callus with revolutionary benefits, especially in the dry technique:

- can be disinfected and sterilized
 - extremely high-cost efficiency due to the durable diamond grit, medCaps can be used more than 300 times
 - gentle, precise concentric running
 - a perfect identification of the diamond grits by coloured dots
 - environmentally friendly
 - easy to handle
- = BUSCH Premium Quality -
foot care at its highest standard

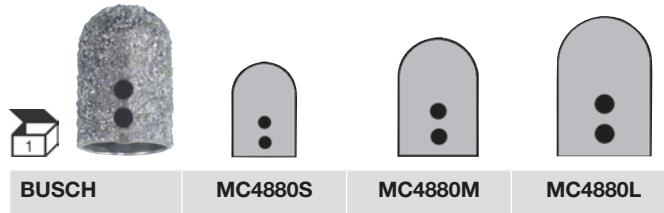
Cap and mandrel have to be disinfected and sterilized unmounted for hygienic and technical reasons.

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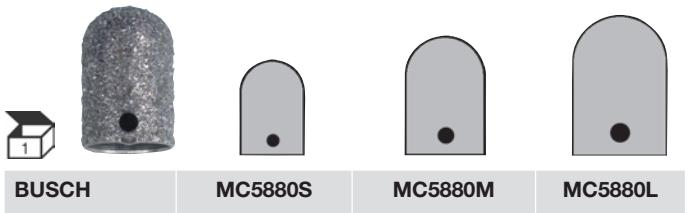
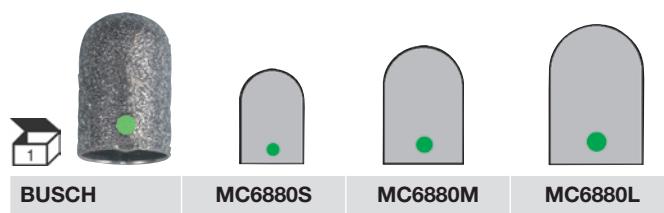
The hygiene pack is easy to integrate into the hygiene circle. In the closed state of the STERI-SAFEpro the medCaps hang loosely on three MC-retaining pins and must separately disinfected and sterilized perfectly without any instrument falling out; if in the autoclave at 134°C or in the hot-air sterilizer at 180°C.

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with passive safety

- only the front end is diamond-coated
- for shortening nails
- smooth polished exterior surfaces
- safety bevel at the edge to the front end
- minimizing risk of injury

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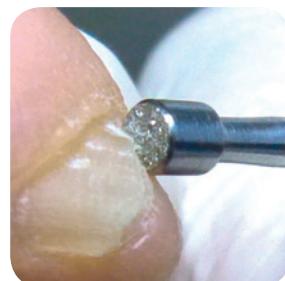
- with passive safety
- smoothly polished front end
- safety bevel on the edge of the front end
- gentle filing right to the nail wall
- minimizing risk of injury

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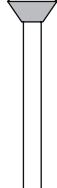
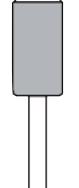
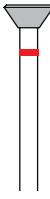
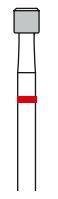
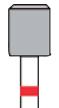
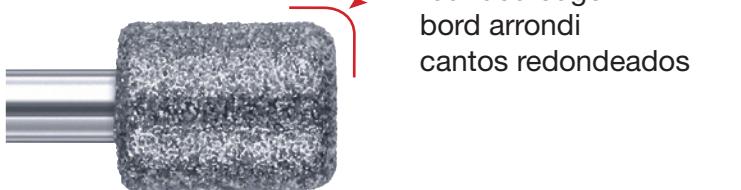
- with passive safety
- smoothening of nails
- removal of callus
- rounded edges
- minimizing risk of injury

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CD-Diamond Instruments

With a peripheral cooling groove technique enabling a cool, soft and pleasant working in both dry and wet technique.

- removal of thick callus
- removal of hard callus

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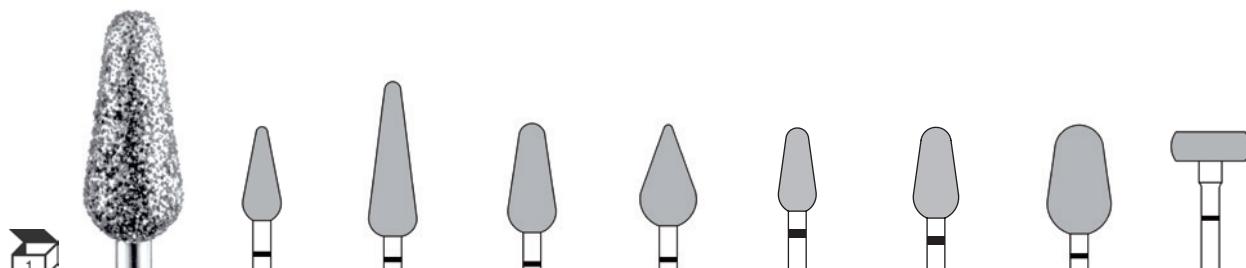
coarse grit

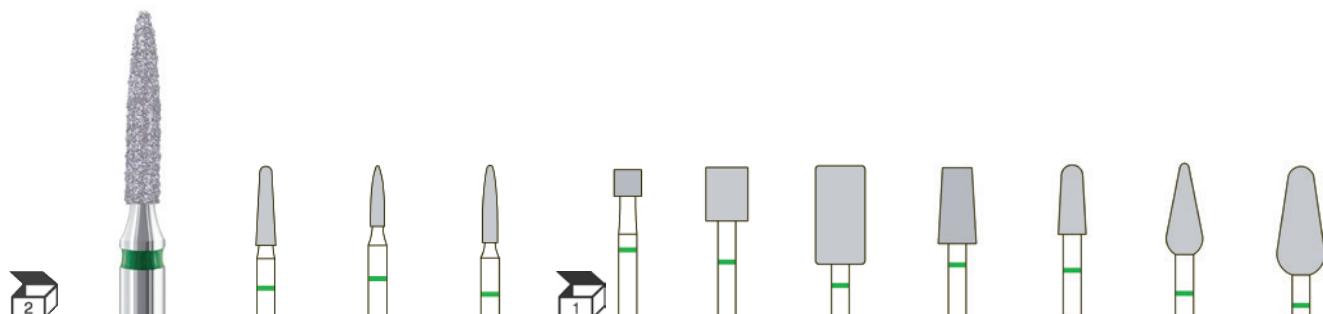
- removal of callus and coarsely nail treatment
- faster abrasion of acrylic and gel

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medium grit

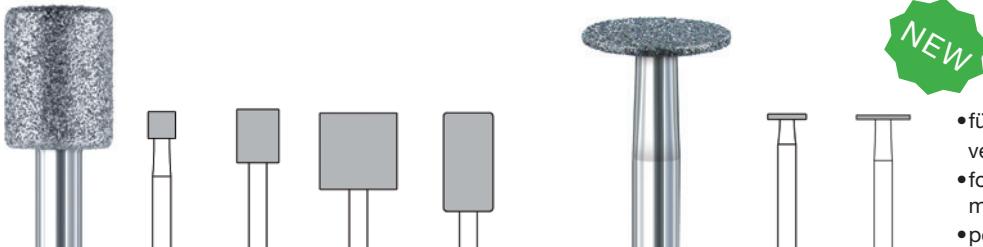
- callus and nail smoothening
- smoothening and refinishing of acrylic and gel

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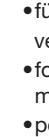
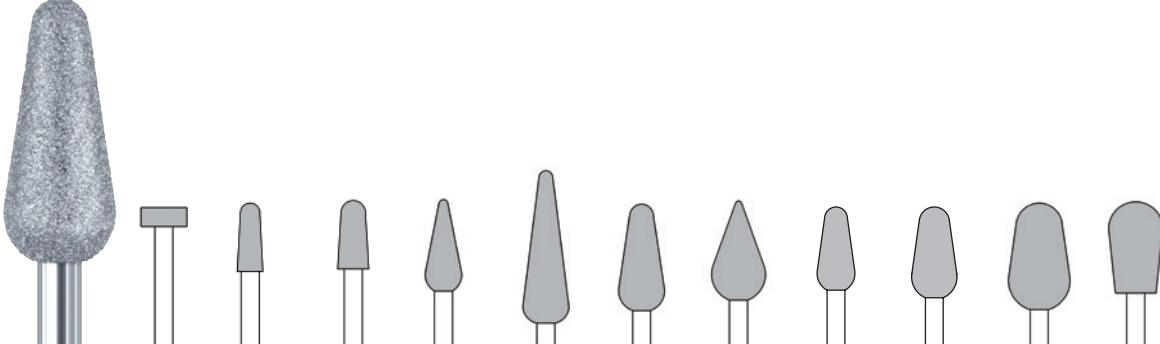
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
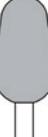
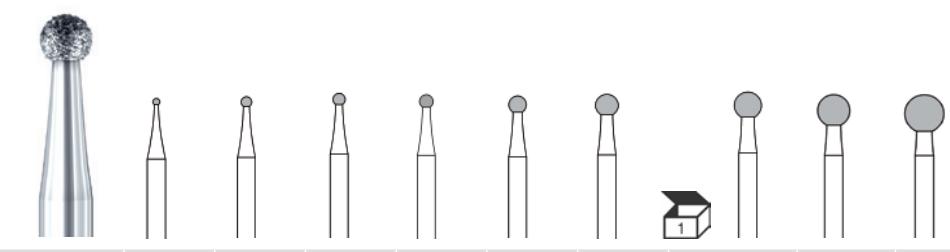
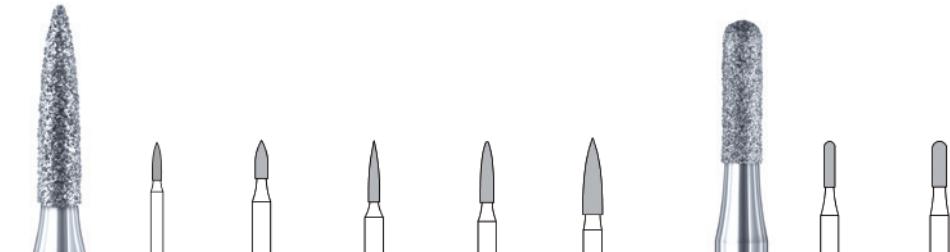
fine grit

- delicate treatment of cuticle and nails

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extra fine grit

- delicate treatment of cuticle, prepolishing of nails

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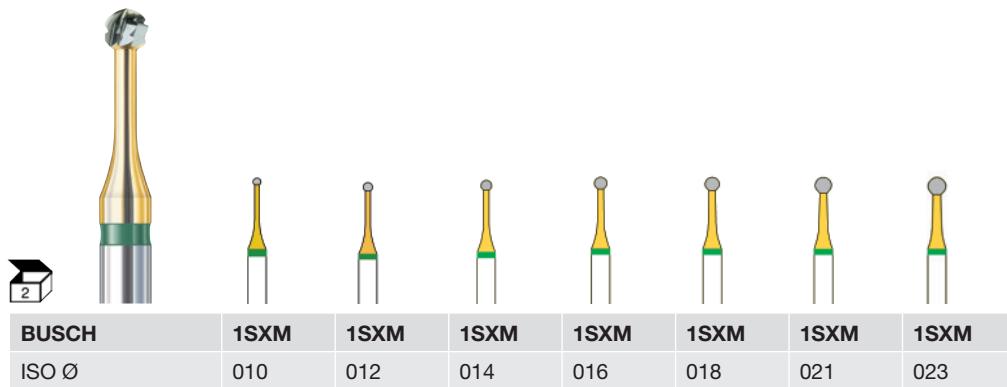
- for a gentle removal of clavi and fungal nail as well as the treatment of grypotic nails and periungual callus
- drilling through gel/acrylic to fix piercings on the front tip of the nail area
- to place deepenings in gel/acrylic so that all kinds of gems, inlaid motifs and stickers can be fixed
- the extremely slender neck enables a better overview of the treatment area
- longevity due to finest grain carbide
- the gold coloured neck and green ring ensures an easy identification

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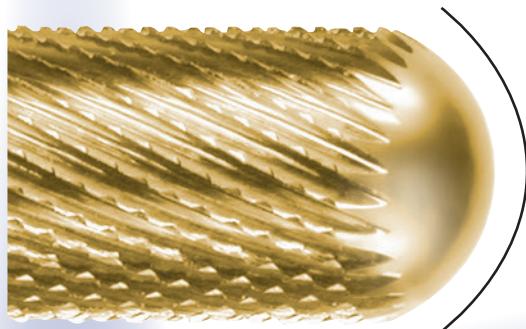
fast - due to the effective SPEED cuts
safe - due to the top of the very smooth working part
 A clever combination with a dual-use.

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With their revolutionary cuts and smooth safe-end working part, the golden T-SPEED series of carbide cutters with TiN coating produces optimum treatment results quickly.

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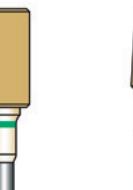
- fast treatment on thick nails and acryl in the nail prosthetics
- treatment of acryl for naildesign

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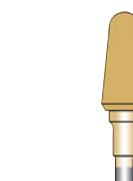
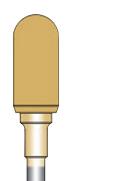
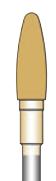
- treatment on thick nails and gel in the nail prosthetics
- treatment of gel for naildesign

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Marking on shanks for a better identification

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- treatment of very thick grypotic nails and hyperkeratosis as well as of orthesis silicone
- for acrylic, gel and natural nail

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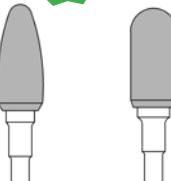
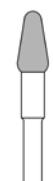
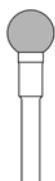
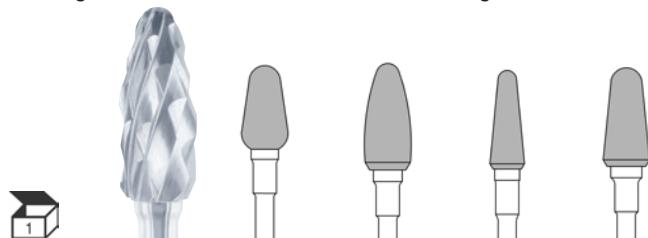
very coarse double cut
the removal of hard nail substance e.g. at wooden nails

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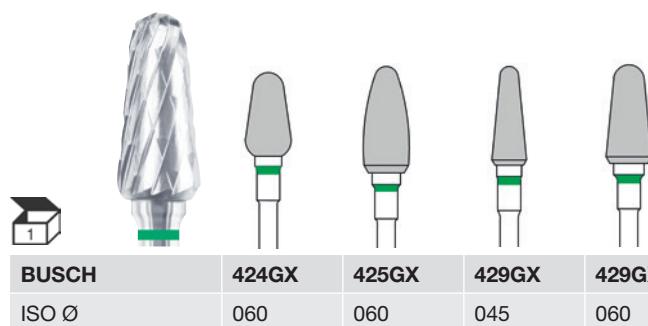
coarse double cut
rough designing at nail treatment

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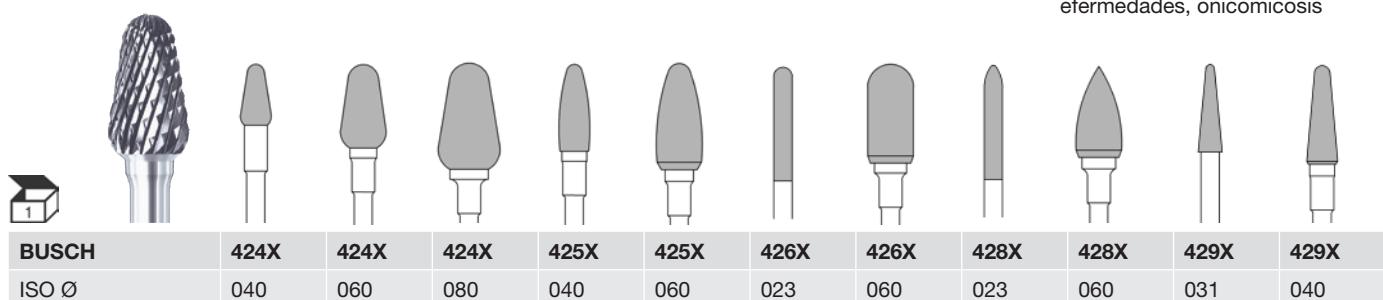
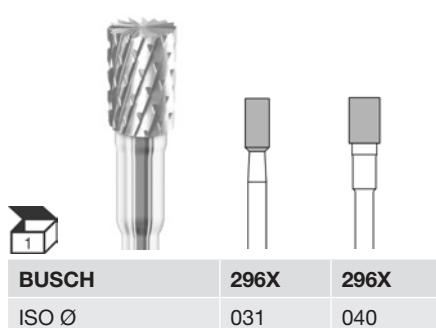
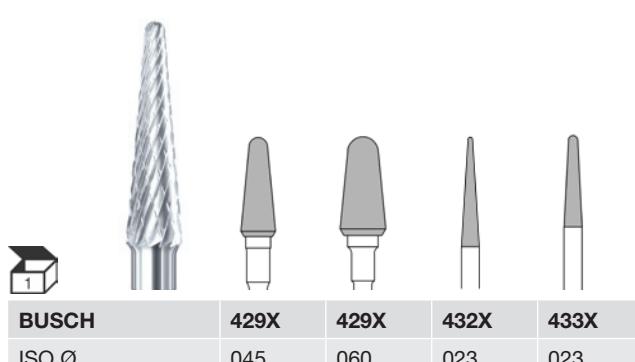
medium double cut
specific outlining e.g. at fungal nails

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fine double cut

delicate treatment of the nail surface before polishing

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Pointed carbide cutter with a special four edged working part.
For finishing and cutting of silicone and orthesis.

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Think smart and benefit from the price advantage of low material and manufacturing costs!
It holds true: The shorter the working part, the higher the price advantage.
You can decide how much you want to save.

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GQSR-cut
coarse straight cut with cross-cut

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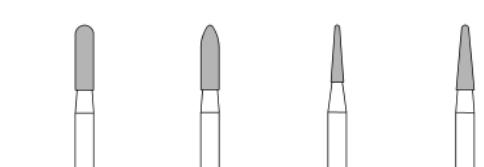
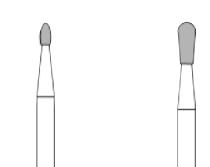
medium double cut (X)

for the treatment as well as the finishing of e.g. fungal nails

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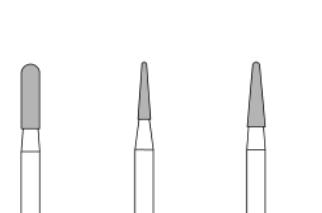
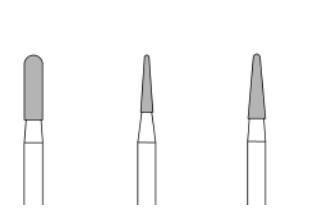
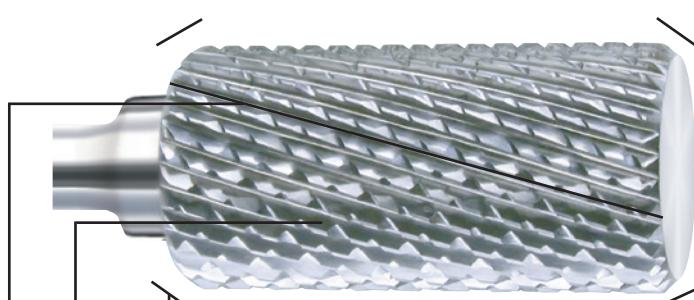
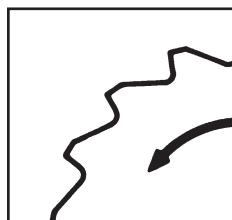
The carbide cutter range SNC was specially developed for the treatment of thickened natural nails as well as artificial nail material for prosthetic corrections. A greater abrasion performance, low vibration running, smooth surfaces, a long service life and a safer handling characterizes this product range.

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SNC safety bevels minimize the risk of injury

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SNC double cut for greater abrasion performance and a smoother running

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SNC background chamfer on the main cut; smooth surfaces and a long service life

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This special high-performance ceramic captivates through excellent fracture toughness and hardness. Its 25% lower weight facilitates smoother cutting, which both patient and user can noticeably experience during the treatment of diseased natural nails or artificial nails. Patients who have a strong metal allergy will benefit from the plus in biocompatibility.

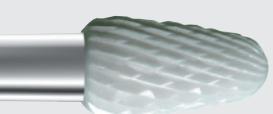
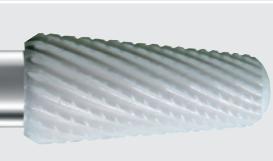
KERA cutters are medical devices and suited for all approved disinfection and sterilization methods.

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coarse straight cut cross cut

- for thick gryphotic nails and rapid removal of orthesis silicone
- with very high abrasion performance for acrylic material

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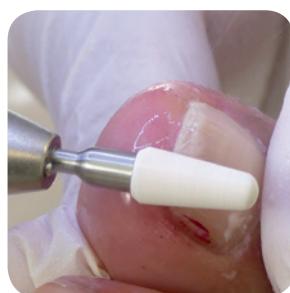
medium double cut

- for specific outlining of misshapen or fungal nails
- with high abrasion performance for gel

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SPEED-cut

- for thick nails, suitable for universal use
- with passive safety for gel and acrylic material

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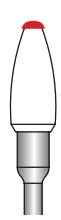
- using of stainless steel ensures an optimal anti-rust protection also after sterilization

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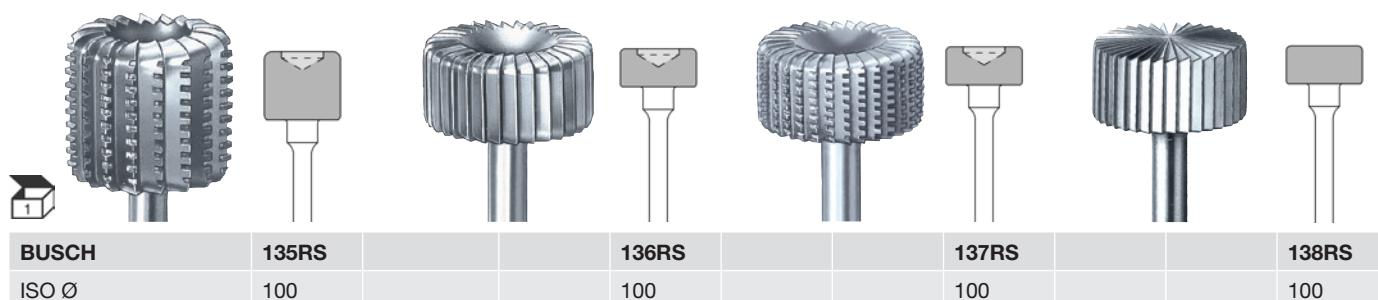
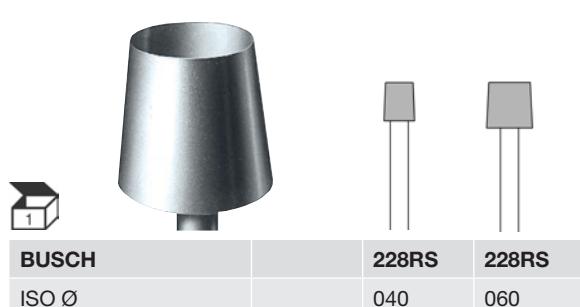
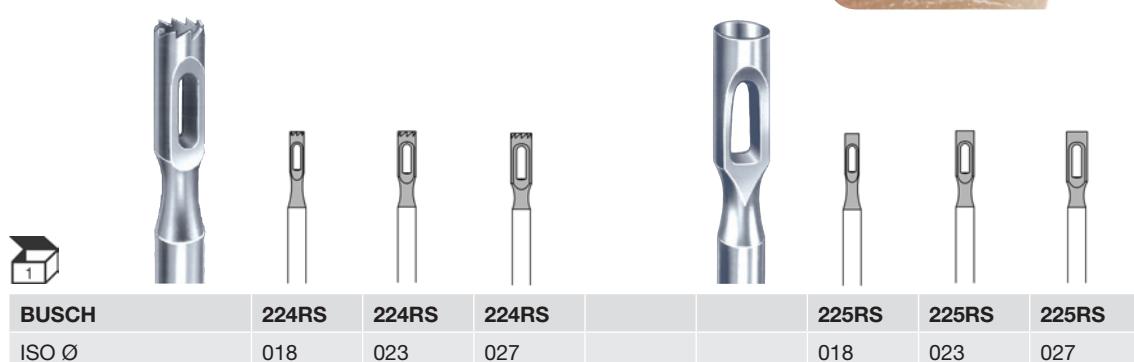
for a gentle removal of corns/clavi
recommended speed 3.000-5.000 r.p.m.

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**polygonal working part
without sharp blades**

- for the gentle removal of deposits in the nail fold at an instrument speed of 5.000 - 20.000 r.p.m.
- for the gentle removal of the cuticle at an instrument speed of 5.000 - 20.000 r.p.m.
- we recommend the version 406L-RS for wide-area work.
- by this application diamond instruments with coarse grit or rather instruments with cuts leave unintentional grooves on the nail

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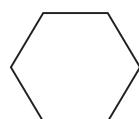
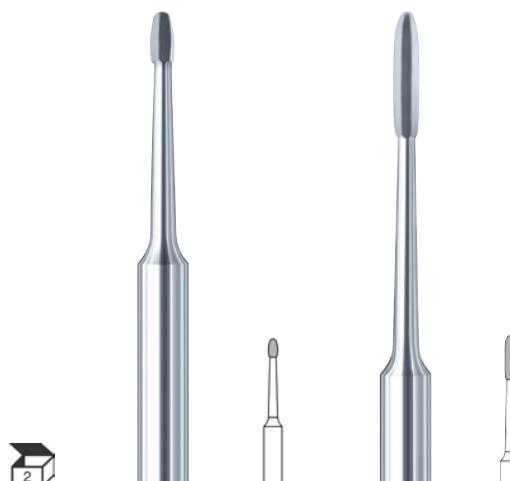
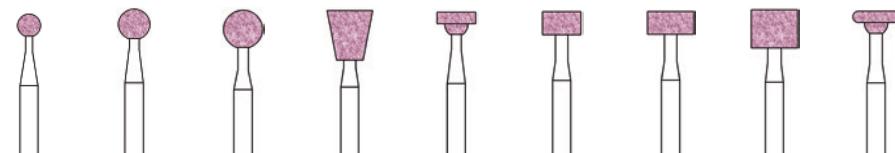
high-grade corundum (Al_2O_3), pink
• for fine grinding work on nails and cuticle

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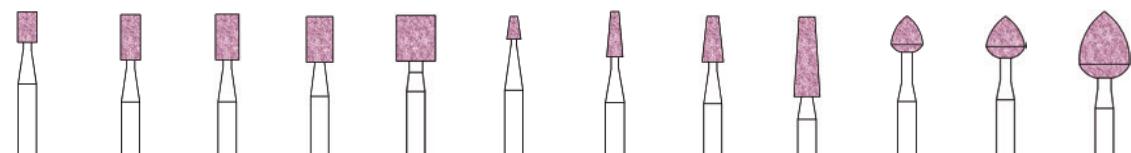
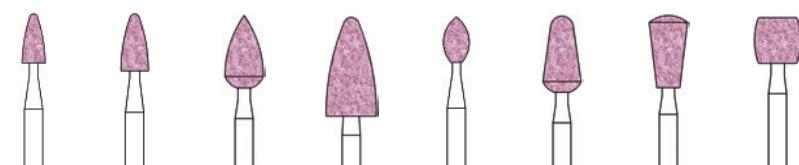
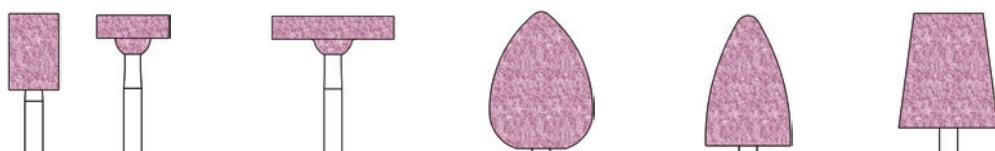
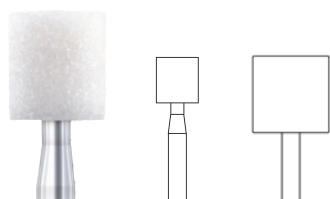
high-grade corundum (Al_2O_3), white
• nail prosthetics and brace technique

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- polishing of the nail surface in nail prosthetics
- polishing of acrylics and nail prosthesis
- fine rounding-off of the nail edges
- smoothening sensitive skin areas
- smoothing of rough surfaces of natural nails and/or gel/ acrylic

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micro fibre-polishing buffs

• final high-lustre polishing after use of paste

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**SOFT WHEELS,
impregnated buffs**

polishing of healthy natural nails as well as artificial nail material (dry technique is recommended)

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pre-polishing, green
corundum (Al_2O_3)
grit size medium

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high-lustre polishing, pink
corundum (Al_2O_3)
grit size extra fine

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brushes

goat hair, sleek
• for the polishing of nails with paste
• for the cleaning of nails

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6

cleaning brush

horse hair, hard
• for the cleaning of steel- and carbide instruments

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Brushes, polishers and polishing buffs are considered „cosmetic products“ in podiatry and therefore do not bear the CE-mark.

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Maße:
100 mm x 25 mm x 14 mm

polishing paste

diamond interspersed paste (3 µm)
• for high-lustre polishing of acrylics and nail prostheses.

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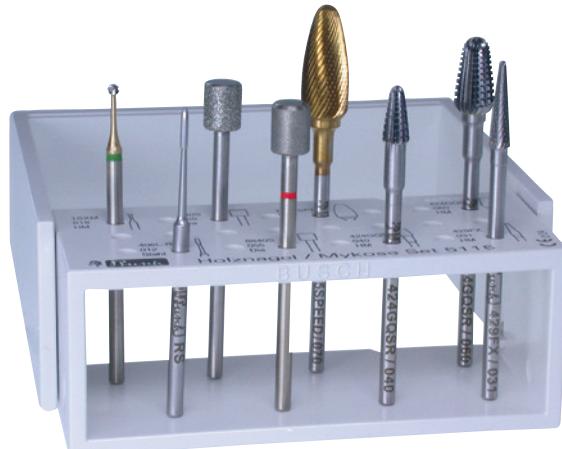
instruments
+ Steri-Safe_{pro}
(for disinfection and sterilization
up to 180°C)
+ hygiene box Podo-Lock
HPL 805 (desinfectable by
spraying or dipping method)

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Thoroughly systematic working
procedures, hygiene and
organization are important
for treating patients safely
with rotary instruments.

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systematic

To support your working procedures,
we have designed instrument sets
for specific applications. The
configuration of these instrument
sets is considered
a recommendation only, and can
be complemented as required.
The sets are delivered in the new
hygiene box PODO-LOCK.

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hygiene

The Steri-Safe_{pro} is a universal
bur holder for storing and
organizing burs as well as being
perfect for disinfecting, cleaning
and sterilizing specific
instruments up to 180 °C.
Please turn to page 48 for
further details about the
Steri-Safe_{pro}.

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organization

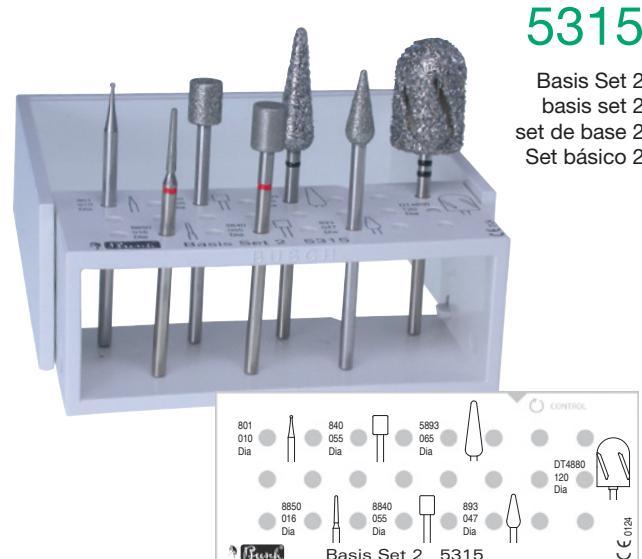
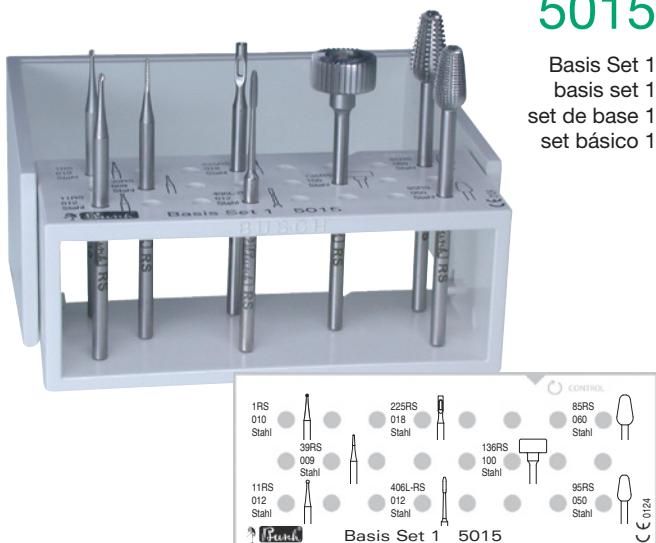
Practical instrument sets, which
are markable, enable the provision
of hygienic prepared instruments,
which after going through the hy-
gienic cycle can be prepared and
put back again for the next use.

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a selection of stainless steel instruments for treating the nail fold, corns/clavi and for removing callus

- 1RS, 11RS, 39RS for treating hard nail folds
- 406L-RS for cleaning the nail fold / removing cuticle
- 39RS for small fissures / removing skin
- 225RS, 11RS, 1RS for removing clavi
- 136RS for grinding mycotic nails
- 85RS for removing callus
- 95RS for smoothening callus

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a selection of diamond instruments in various grit sizes for gently grinding callus and nails

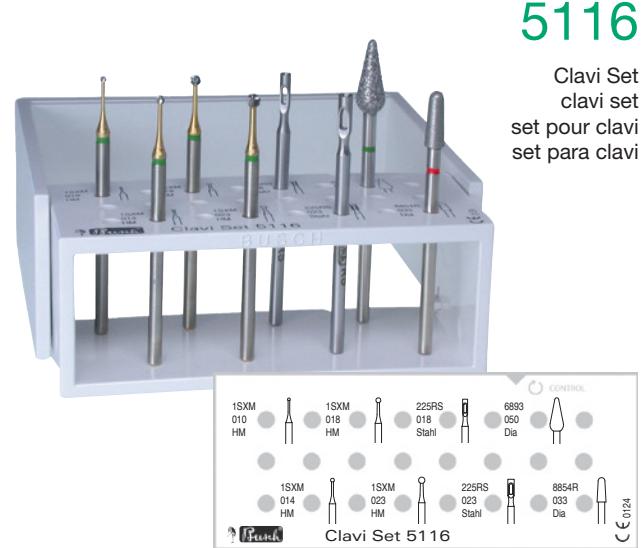
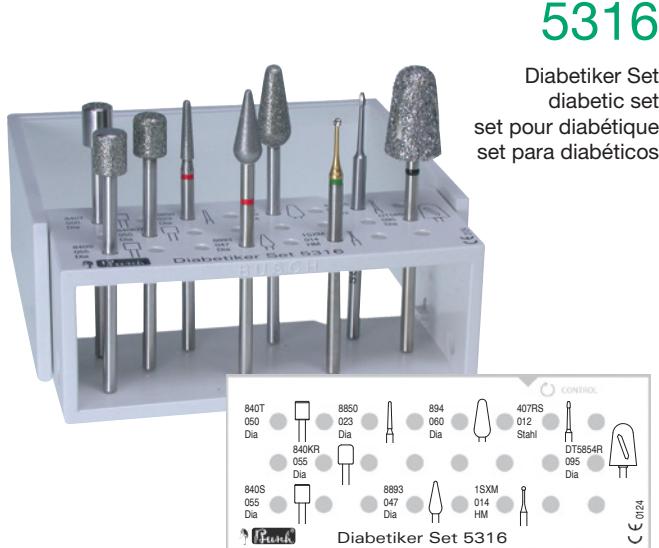
- DT4880 for removing thick callus
- 5893 for removal of major callus
- 893, 840 for smoothening callus
- 801 for removing clavi
- 840 for smoothening nails
- 8840, 8850 for fine treatment of cuticle and nails

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for minimum invasive treatment of the diabetic foot; rounded instruments with only partial diamond coating minimize the risk of injury during treatment

- DT5854R for removing major calluses
- 840KR, 894 for smoothening callus
- 840S (SIDE GRIP series) for safe nail treatment
- 840T (TOP GRIP series) for shortening nails
- 8893, 8850 for fine treatment of cuticle and nails
- 407-RS for cleaning the nail fold / removing cuticle
- 1SXM for clavi, as well as subungual and periungual calluses

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a user-oriented selection of instruments for specifically treating and removing corns

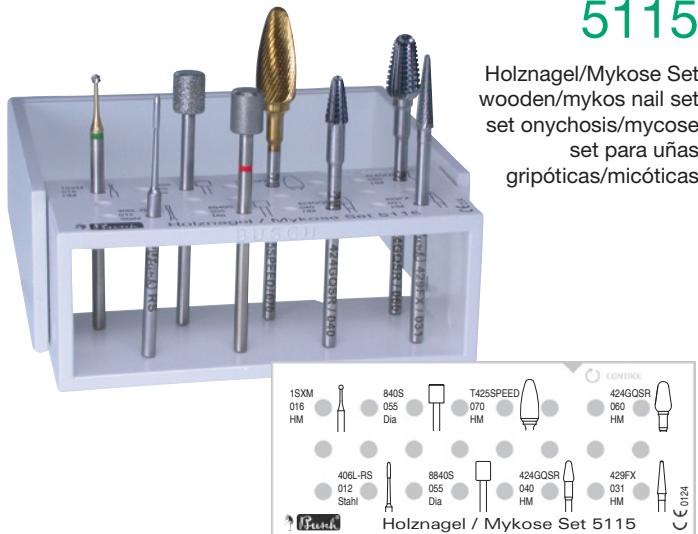
- 1SXM for gentle grinding of clavi and mycotic nails
- 225RS special treatment for clavi
- 1SXM for peringuinal calluses
- 6893 for treating calluses
- 8854R for smoothening cuticle

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the ideal basis set for precise, quick yet gentle treatment of mycotic / gryptic nails and for removing large quantities of nail substance

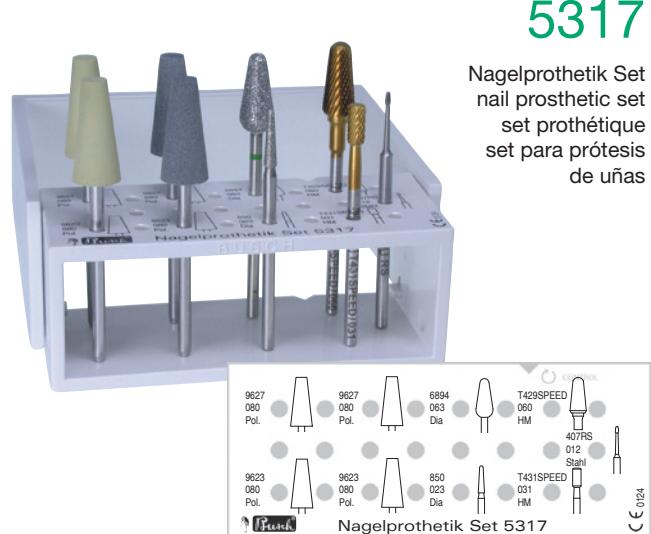
- T425SPEED for working on thickened natural nails
- T425SPEED for working on artificial nail material
- 424GQSR for working on gryptic nails
- 429FX for fine treatment of nails
- 840S (SIDE GRIP series) for working on mycotic nails
- 8840S (SIDE GRIP series) for fine treatment of mycotic nails
- 406L-RS for cleaning the nail fold / removing cuticle
- 1SXM for clavi, as well as subungual and periungual calluses

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for specific treatment of diseased natural nails and artificial nail material in nail prosthetics

- 6894 for coarse nail treatment
- 850 for forming the nail around the nail fold
- 407L-RS for cleaning the nail fold / removing cuticle
- T429SPEED for working on thickened large nails
- T431SPEED for working on thickened small nails
- T429SPEED, T431SPEED for working on artificial nail material
- 9627 for prepolishing nails
- 9623 for high-lustre polishing of nails (with coolant)

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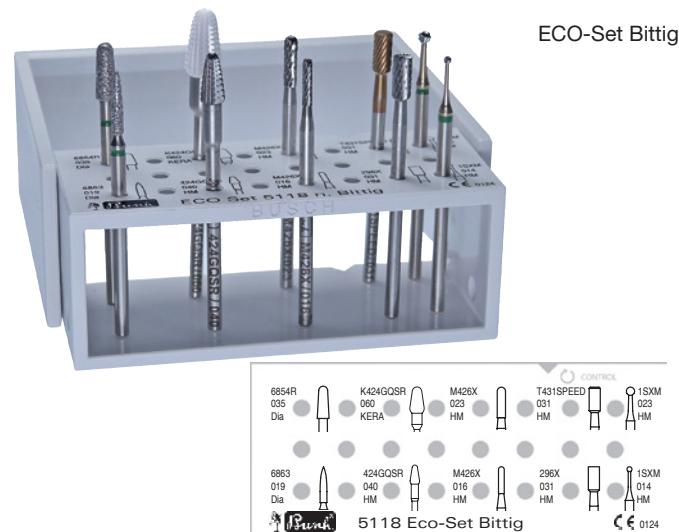
Sterilization of polishers max. up to 134°C.

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for all standard jobs on cuticle and callus, even on a diabetic foot

- HT6880 for removing major calluses
 - 894, 5894 for treating chapped skin and hyperkeratoses
 - 5840, 5894 for thickened, diseased nails
 - 840, 894, 850, 854 for shaping and smoothening nails
- further information can also be found in the pictorial atlas of medical podiatry according to Bittig, 3rd edition

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for Bittig grinding and cutting technology for skin and nails

- removal of clavi and nail mycosis by means of cutting 1SXM
- gentle removal of mixed substances (skin and nail) 295X, T431SPEED
- removal of mycotic nail substance M426X
- quick removal of thickened nails and hyperkeratosis 424GQSR, K424GQSR
- removal of callus in the nail fold area 6893, 6854R

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The brush and polishing-buff set is thought to be for cleaning and for the final finishing by working at a low speed.

- the nail surfaces are precisely and effectively cleaned from the annoying visual blockage from the grinding dust with the cup-shaped goat hairbrush 9739
- the same soft wheel-shaped goat hairbrush is especially suitable for the perfect dusting and cleaning of the edge areas 9738
- the finishing for the high-lustre polishing can be done with the 4-layered micro fiber polishing-buff 9644
- at a low speed, the horse hairbrush can be used for the gentle cleaning of your rotary instruments 9736
- Brushes and polishers are not medical products

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for sterilization up to 180 °C
 • for an individual and clear
arrangement of instrument sets.
 • for the safekeeping of
instruments before, during or
after disinfection, cleaning and
or sterilization.
 • for all usual methods of disinfection,
cleaning and sterilization
(see basic information).

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All instrument sets on the
STERI-SAFEpro are supplied
with the hygiene box HPL 805.

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material: polystyrene
colour: ivory

double stand 5762:

possible assortments:
29 holes HP + 12 holes FG
dimensions:
33mm x 66mm x 58mm

quadruple stand 5764:

possible assortments:
62 holes HP + 24 holes FG
dimensions:
33mm x 66mm x 58mm

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socle double 5762:

assortiment possible:
29 orifices PM + 12 orifices FG
dimension:
33mm x 66mm x 58mm

socle quadruple 5764:

assortiment possible:
62 orifices PM + 24 orifices FG
dimension:
66mm x 66mm x 58mm

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shank FG Ø 1,60 mm

- only for the use in turbines with cooling water

Note the maximum permitted rotational speed (specified on every BUSCH package).

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mittlere Körnung

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CD diamond instruments

- with peripheral cooling-groove technique
- enabling a cool, soft and pleasant working

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medium grit
callus and nail smoothening

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a selection of FG instruments for use in the turbine with water cooling, for specific treatment of clavi, mycotic nail surfaces and fissures, as well as for smoothening callus and nails

- CD06, 1S for removing clavi
- CD80 for filing and forming callus and nails
- CD54, CD39 for working on mycotic strips
- 851 for filing mycotic nails and fissures
- 909 for smoothening callus and nails

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material: medical resin

colour: light grey

- for an individual and clear arrangement of instrument sets.
- for the safekeeping of instruments before, during or after disinfection, cleaning and/or sterilization
- for all usual methods of disinfection, cleaning and sterilization (see page 59)
- with closed handle the instruments are protected from falling out
- colour-coding due to non-slip silicone strips

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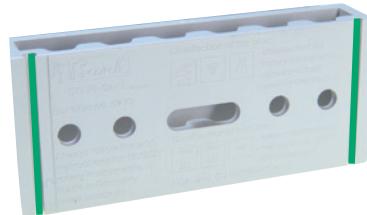
Maße:
100 mm x 25 mm x 14 mm

dressing stone

high-grade corundum, ceramic bonded
 • for the removal of particles sticking to diamond instruments at an instrument speed of 1.000-5.000 rpm.
 • dampen before use

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Maße:
120 mm x 50 mm x 20 mm

dressing stone

silicon carbide, ceramic bonded
 • for dressing polishers and the preparation of clogged high-grade corundum abrasives at an instrument speed of 10.000 r.p.m.
 • in 2 grit sizes, fine and medium

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Maße:
120 mm x 12 mm x 6 mm

dressing diamond

• for the dressing of deformed abrasives at an instrument speed of 10.000 r.p.m.

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Maße:
20mm x 22mm x 4mm

ceramic block

coarse grit
 • for dressing of polishers and abrasives

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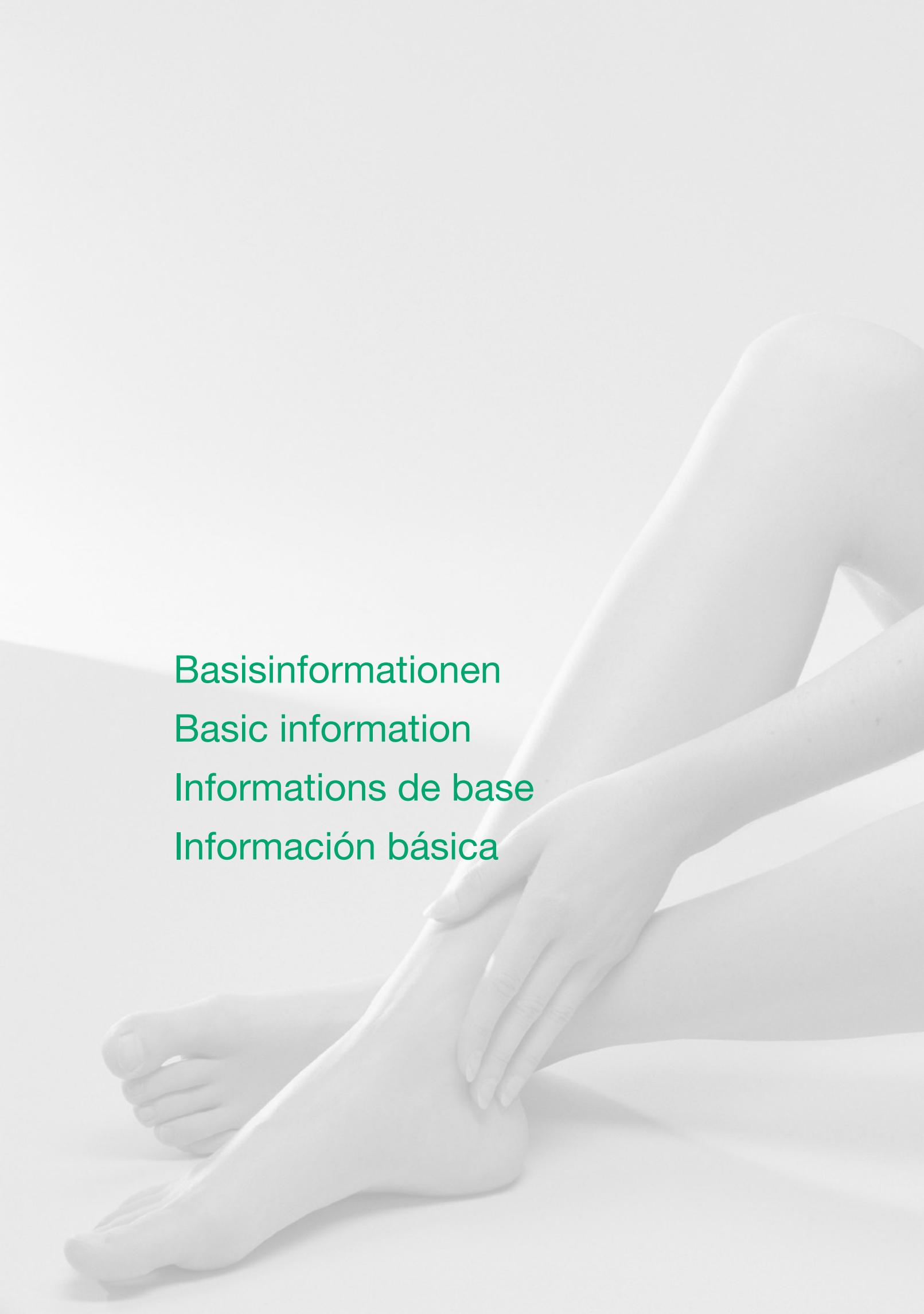
Medical devices legislation for medical products (CE marking)

On basis of the European Medical Device Directive 93/42 EEC (MDD) Germany introduced the Medical Product Law in 1994 (with a 5-year interim period). This law and the regulations attached to it have to be applied by the developer, manufacturer, distributors and user of rotary instruments in the medical field. The demands on the medical products and on the quality management system that were formulated in the law or rather in the EC-guidelines are being fulfilled by BUSCH & CO. GmbH & Co. KG since 1998, by the certified proof of the procedures according to the MDD. At the same time the requirements of the quality system standards ISO 13485 are being successfully converted and confirmed yearly by certification audits. All BUSCH instruments for medical pedicure/podiatry are medical products in the sense of the MDD, which are as such to be noticed by the CE marking as a conformity proof on the label of each package. In the field of podiatry brushes and polishers are considered as cosmetic products - because of this they do not bear a CE marking.

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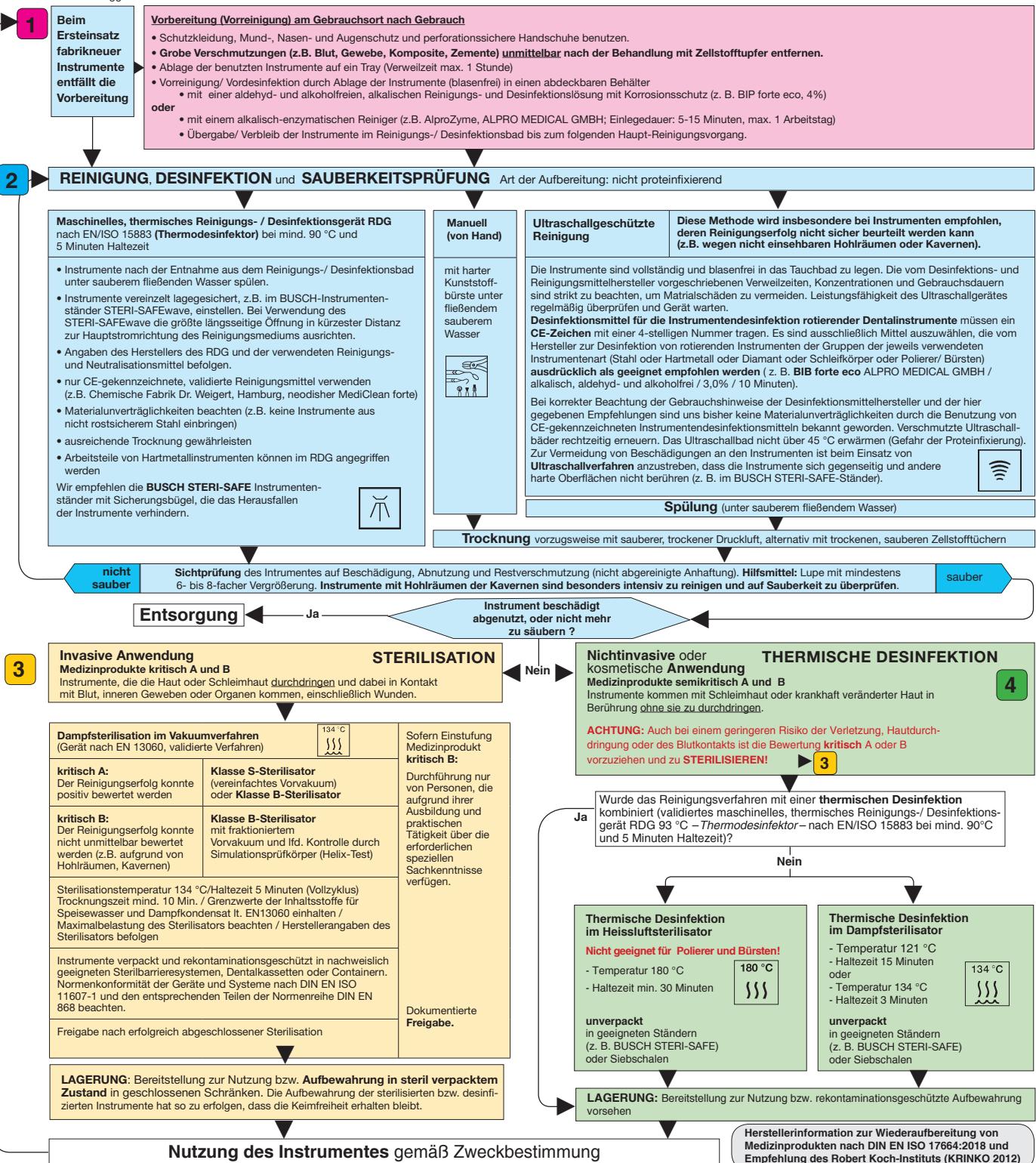
Safety recommendations

- Until they are first used, rotating instruments should be kept in their original packaging, and protected from dust and moisture, at room temperature.
- Always keep the packaging (also during the active usage period) so that the instruments are traceable if required.
- Always use fully functioning, correctly serviced and maintained as well as cleaned turbine drives as well as hand-held and angle-piece drives.
- Insert the instruments as deeply as possible. Check for firm seating.
- Use respiratory and/or eye protection and an extraction system.
- Before starting to work on the workpiece, bring the instruments up to operating speed and make sure that they are running concentrically.

- Observe the maximum speed (as indicated on all BUSCH packaging). Risk of overheating and injury.
- Make sure that the instruments don't get jammed or levered.
- Do not exceed a contact pressure of 0.3 to 2.0 N. (Risk of breakage and iatrogenic injuries in the working space and danger of heterotrophic ossification after overheating/burning.)
- Only use the instruments in accordance with their intended use.
- Failure to comply with or adhere to the following hygiene recommendations can lead to transmission of pathogens.
- If possible, use the entire length of the working part in order to avoid point overheating, e.g. of the tips (resulting excessive mechanical stress and local overheating).
- Make sure that there is sufficient water cooling to avoid unwanted heat development (overheating). For the dental practice, this means a minimum cooling flow of 50 ml per minute ensure a minimum.
- FG instruments with a total length of more than 22 mm or a head diameter more than 2 mm might require additional cooling.
- Surgical instruments with a long shaft might require additional cooling.
- Bent or non-concentrically running instruments or instruments with damaged or worn working parts must be rejected and disposed of in order to avoid injury or overheating through friction caused by blunt instruments.

Hygiene recommendations

1 Preparation

2 Cleaning

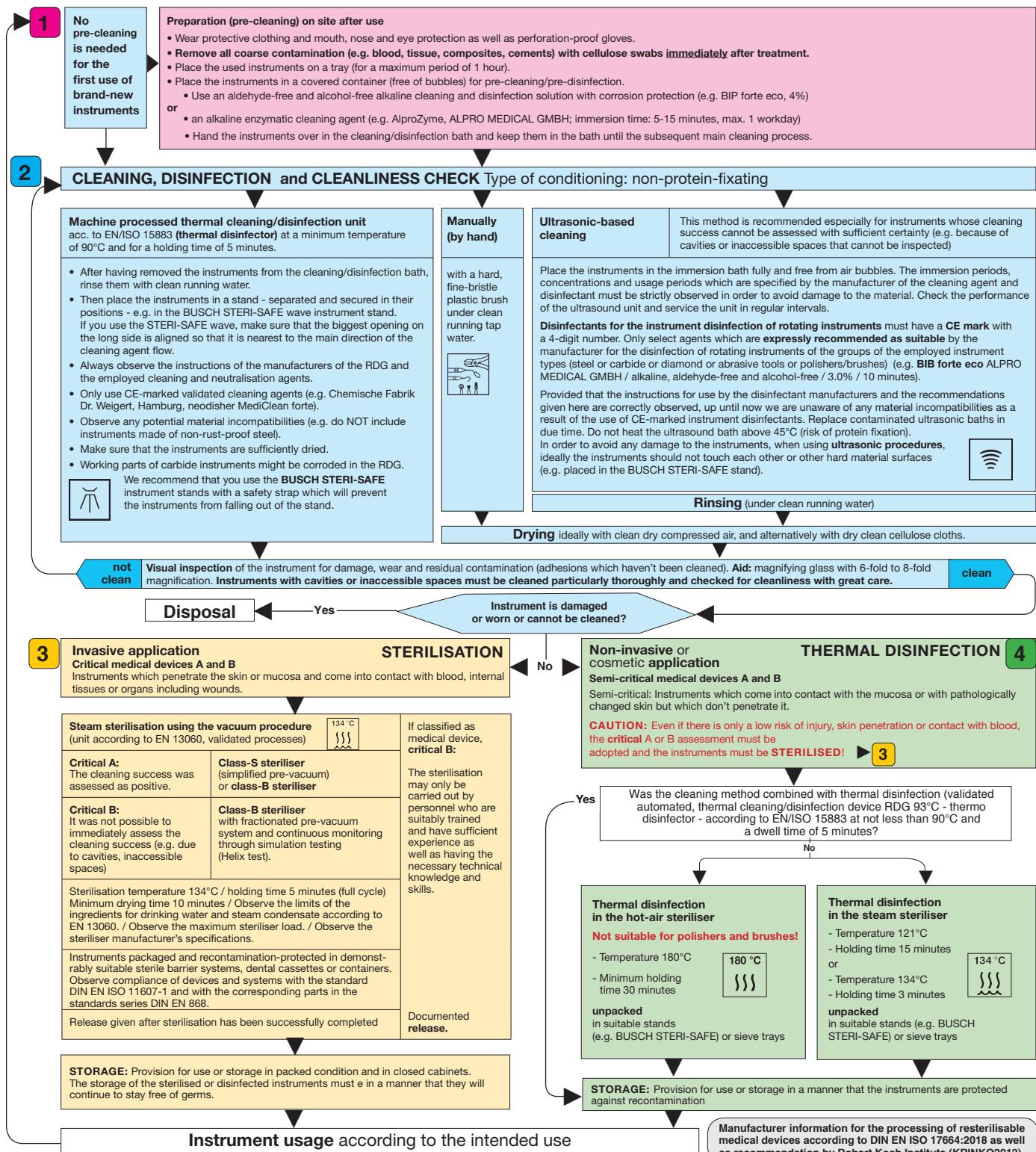
3 Sterilisation

4 Thermal disinfection

Area of application: Rotating instruments made of steel, carbide, ceramic or diamond as well as polishers, abrasive tools and brushes that are intended for the application on humans. The instruments are delivered non-sterile. Before they are used for the first time as well as after each use, they must be disinfected or cleaned and disinfected or, if required, sterilised.

Grinding caps and cap mandrels have to be disinfected and sterilized unmounted for hygienic and technical reasons.

Restrictions for the processing of resterilisable instruments: Brushes for prophylaxis are single-use products because it cannot be guaranteed that the brushes can be cleaned entirely free from residue. For non-rust-free instruments, disinfectants and cleaning agents with corrosion protection must be used. **Non-rust-free instruments are not suitable for steam sterilisation.** The product service life is determined by wear and damage caused by the instruments' use. Therefore, specific details regarding the number of processing runs cannot be provided. Never use hydrogen peroxide (H_2O_2) for disinfection of the instruments because the risk of damage to the material cannot be excluded.

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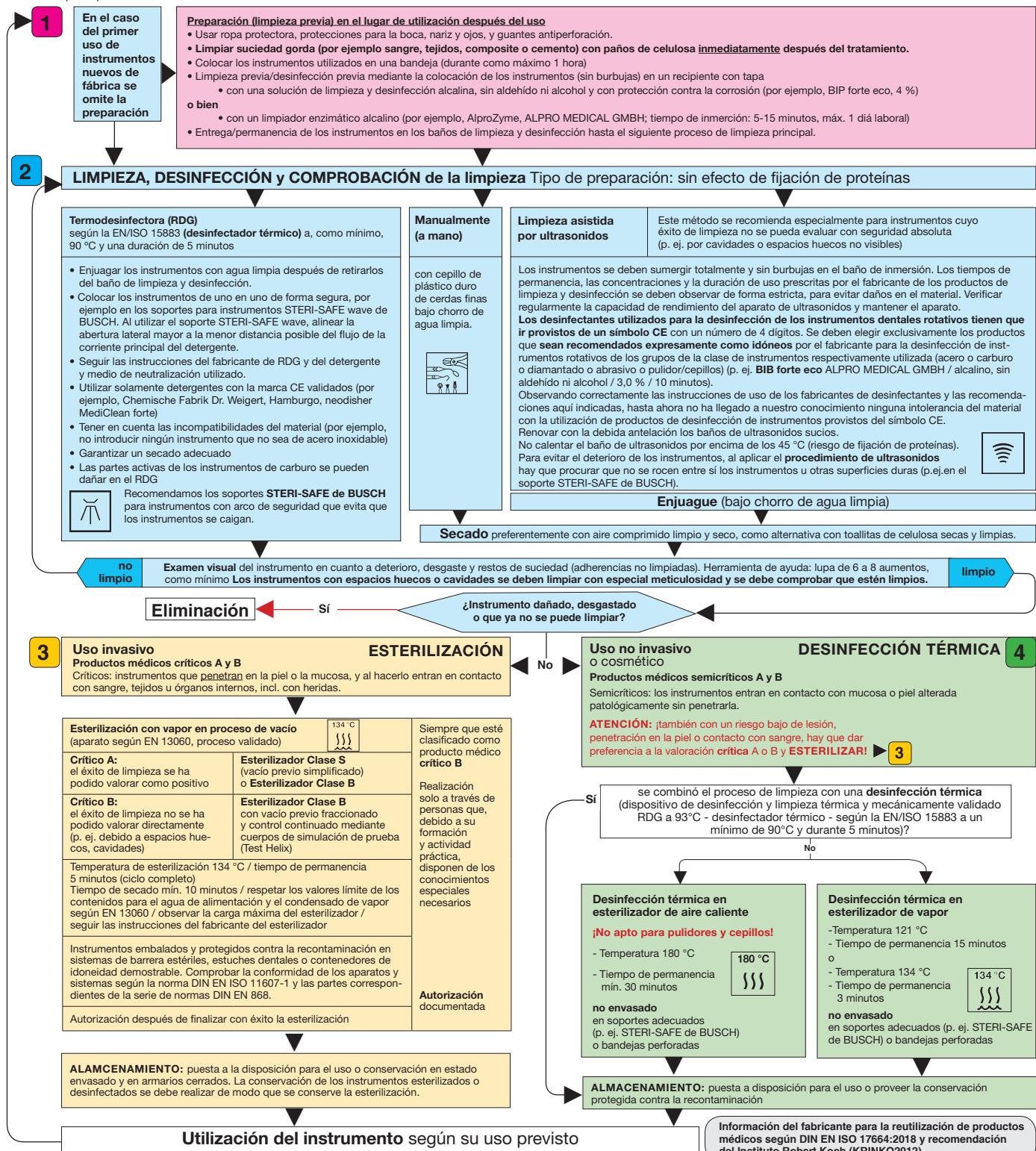
max. admissible speed

The mentioned maximum admissible speed (rpm) indicated on each BUSCH pack is not allowed to be exceeded due to safety regulations.

recommended speed for use in micromotor

To produce optimum results, run the rotary instruments at their recommended speed. See diagram. The recommended speed must be adapted in line with the respective applications and medical circumstances. Lower speeds should be used when working in the dry technique in order to reduce the risk of heat building up. If the diameter of the working part exceeds that of the shank, powerful centrifugal forces may build up at higher speed which might bend the shank and/or fracture the instrument.

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technical data

HybridTWISTER/HybridCap

working part: mixed grain adapted to applications, galvanic bonded.
shank material: stainless steel

diamond Instruments/medCap

working part: diamond grit adapted to applications, galvanic bonded.
shank material: stainless steel.

abrasives

working part: selected High grade corundum, grit size adapted to applications and diameter of abrasives, ceramic bonded.
shank material: stainless steel.

carbide-Instruments

working part: carbide, finest grain quality
shank material: stainless steel

ceramic cutters

working part: high performance ceramic
shank material: stainless steel

steel Instruments (RS)

working part and
shank material: stainless steel

polishers

working part: silicone bond mixed with abrasive materials
shank material: stainless steel

Brushes/Polishing buffs/ SOFT WHEELS

material: bristles/micro fibre
shank material: stainless steel

shaft diameter

HP 2,35 mm (-0,016)
FG 1,60 mm (-0,010)

total lengths

HP 41,5 mm - 55 mm
FG 19,0 mm - 24 mm

working part diameter

0,5 - 25,0 mm

concentricity tolerance above standards

standards

DIN, ISO, EN
maximum admissible speed mentioned on each package.

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Important information:

- Select appropriate qualified cleaners/ disinfectants giving special regard to material friendliness; do not use any household detergents; do not use acids or alkaline solutions

- Use consistent dosing (no overdose) according to the manufacturer's specifications; take evaporation into account when using cleaner/disinfectant baths for several days (not only water evaporates, the concentration of chemical ingredients will increase).

Ready-to-use solutions are preferable compared to concentrates.

- Comply with the maximum permissible dwell times in cleaners/disinfectants according to the manufacturer's specifications.

Failure to observe this warning may result in corrosion of the instruments, which will affect their performance, stability and surface texture requiring them to be discarded prematurely.

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BUSCH & CO. GmbH & Co.KG